




## 1 x Abdominal model A-S-A-Evacuable-002+

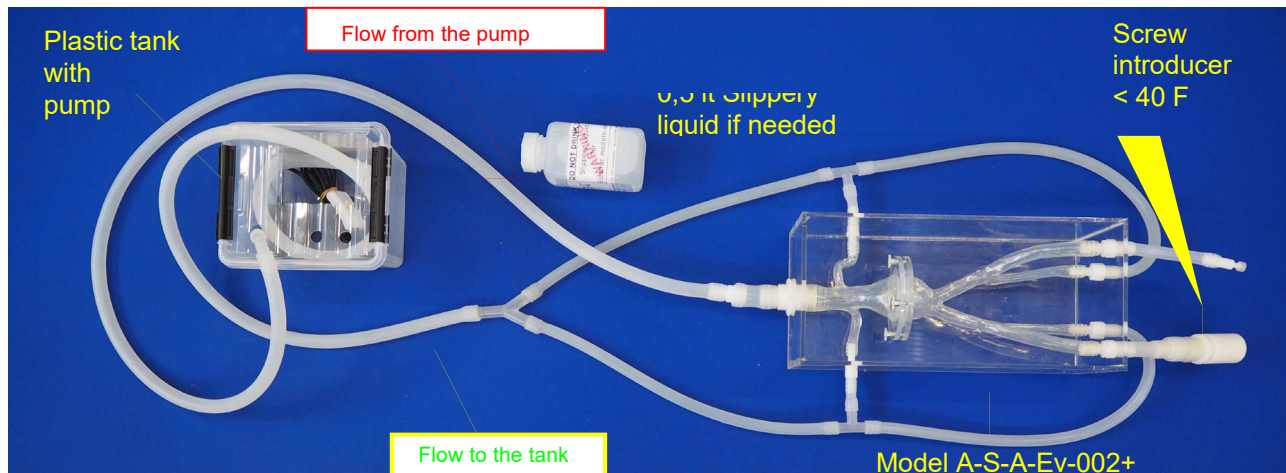

### Slippery liquid:

When the model is ready to work, please insert 30% to 50% of the slippery liquid concentrate bottle into the plastic tank, plug the pump on and wait approx. 15 minutes until the liquid is mixed inside the model. The model is now ready to work.

### How to take your device out of the model:

Unplug the pump, elevate the model with its tubes so all the water goes back inside the tank. Open the model with the 4 screws; take your device carefully out of it. Be careful with screwing the model back together: on the opened aneurysm you have four blue male-female that need to fit together before you screw the aneurysm together. When done, the model is again ready to work.

Have a nice work with this model.

**Swiss Quality Models manufactured by Elastrat in Geneva, Switzerland.**